**WellMag<sup>™</sup>**

**WellMag®**  
*Portable magnetotherapy*

## USER MANUAL - Instructions -

**WELLSTORE srl**

REGISTERED OFFICE: via Italia, 50 20900 Monza (MB) - P. Iva 05907450968  
OPERATIONAL HEADQUARTERS: WELLSTORE srl c/o Genimpex - via Matteotti 128 20864  
Agrate Brianza (MB), Italia  
Tel: 0774-1925266

This document is of ownership of Well Store srl. All the rights are reserved. It is forbidden the copy and the reproduction by any means, included the total or partial photocopy of the content without written authorization of Well Store srl.

<b>CAP1 - PRESENTATION</b>	4
<b>1.1 - What is WELLMAG</b>	4
<b>1.2 - What is WELLMAG for</b>	4
<b>1.3 - Who can use WELLMAG</b>	4
<b>CAP2 - INDICATIONS AND CONTRAINDICATIONS</b>	5
<b>2.1 - Indications</b>	5
<b>2.2 - Controindications</b>	5
<b>CAP3 - BASIC ENDOWMENT AND ACCESSORIES</b>	5
<b>3.1 - Basic endowment</b>	5
<b>3.2 - Accessories (optional)</b>	5
<b>CAP4 - OPERATION</b>	6
<b>4.1 - Connection and application of solenoids</b>	7
<b>4.2 - WELLMAG operating with band EasyCare</b>	7
<b>4.3 - Power source</b>	7
<b>4.4 - Turning on the device</b>	7
<b>4.5 - Program selection</b>	7
<b>4.6 - Start of the therapy</b>	9
<b>4.7 - Setting the frequency in preset low frequency programs</b>	9
<b>4.8 - Setting the intensity in preset low frequency programs</b>	9
<b>4.9 - Setting Time</b>	9
<b>4.10 - End or interruption of the session</b>	10
<b>4.11 - Turning off the device</b>	10
<b>4.12 - Symbols on display</b>	10
<b>4.13 - Verification of the solenoids operation</b>	10
<b>CAP5 - FREE PROGRAMS</b>	11
<b>5.1 - Setting of free program</b>	11
<b>CAP6 - APPLICATIONS</b>	12
<b>6.1 - Therapy planning</b>	12
<b>6.2 - Position to be maintained during the sessions</b>	12
<b>CAP7 - POWER SUPPLY</b>	17
<b>7.1 - Charge indication</b>	17
<b>7.2 - Battery recharge</b>	17
<b>7.3 - Battery maintenance</b>	18

<b>7.4 - Battery replacement</b>	18
<b>7.5 - Precautions for use</b>	18
<b>CAP8 - ERROR MESSAGES</b>	18
<b>CAP9 - MAINTENANCE</b>	19
<b>9.1 - Solenoids</b>	19
<b>9.2 - Device and power supply</b>	19
<b>9.3 - Power supply replacement</b>	19
<b>9.4 - Immediate maintenance</b>	19
<b>CAP10 - WARNINGS</b>	20
<b>CAP11 - TECHNICAL FEATURES</b>	21
<b>11.1 - Power supply</b>	21
<b>11.2 - Output features</b>	21
<b>11.3 - Other features</b>	21
<b>CAP12 - SYMBOLS</b>	22
<b>CAP13 - BIBLIOGRAPHY</b>	22
<b>CAP14 - OPTIONAL ACCESSORIES FOR HF PROGRAMS</b>	23

**ATTENTION:**  
READ CAREFULLY THE INSTRUCTIONS BEFORE USE

WELLMAG belongs to a new line of electromedical instruments for physiotherapy. Its main characteristics are its reduced dimensions, its easy of use and its versatility.

### 1.1 - What is WELLMAG?

Thanks to the constant investigation and research, we created a new system for magnetotherapy, our WELLMAG. Through the preset and ready to use programs and its innovative software, magnetic fields are created for therapeutic purpose and for the treatment of common pathologies (lesions, accidents, illnesses). It's an extremely innovative product thanks to its technological features and its ease of use.

### 1.2 - What is WELLMAG for?

With WELLMAG it is possible to apply magnetic fields with beneficial effects on the reconstruction of the bone tissue and, in general, on the regeneration of damaged tissues. Also, it can be used to obtain analgesic, anti-inflammatory and anti-edema effects.

### 1.3 - Who can use WELLMAG?

WELLMAG expresses its own potential in the medical field (physiotherapy particularly). Nevertheless, thanks to its ease of use and versatility, it can be used not only by physicians and therapists, but also from whoever desires to use it at home. Thanks to its reduced dimensions and rechargeable battery, it can be used anytime and during travelling. **Do not use the product in high-temperature and humid environments.**

**ATTENTION: for home use, please follow the prescriptions of magnetotherapy of your doctor.**

## CAP2 - indications and contraindications

### 2.1 - Indications

The magnetic fields can treat the most common pathologies, mainly the ones related to the bone system and joints. They are especially suitable in the treatment of:

- Recent fractures;
- Pseudarthrosis, arthrosis, arthritis;
- Inflammatory and degenerative arthropathies;

- Osteoporosis;
- Cicatrizations, bedsores;
- Edemas, arteropathies;
- Psoriasis.

Please consult your doctor for diagnose and follow up of the therapy.

### 2.2 - Contraindications

The side effects of the magnetic fields are the same as the ones of physical therapies that act with the endogenous production of heat:

- presence of metallic fragments;
- neoplasms (tumors);
- tuberculosis
- acute inflammation;
- cutaneous lesions;
- sensitivity alteration;
- pace-maker;
- pregnancy;

## CAP3 - Basic endowment and accessories

### 3.1 - Basic endowment

- Device
- Power supply
- 1 pair of solenoids (for LOW FREQUENCY therapies)
- 2 elastic bands
- Magnet for the verification test in low frequency
- 1 band Easy Care (for HIGH FREQUENCY therapies)
- User manual
- Carry bag

### 3.2 - Accessories (optional)

#### For LOW FREQUENCY

- Pair of solenoids
- Elastic bands
- Magneto Belt (25x67) cm
- Pillow for magnetotherapy (50x50) cm

#### For HIGH FREQUENCY

- Accessories for high frequency (*Cap. 14*).

- 1 Display LCD
- 2 (I) = ON/OFF: turn on / turn off
- 3 Cursor up/down
- 4 (+)/(-) : intensity regulation
- 5 Output for applicators cable
- 6 Cable Output

**NOTE:** before applying the magnetotherapy, check the contraindications and consult the physician that prescribed the therapy.

#### 4.1 - Connection and application of solenoids

Connect the cable of the solenoids to the output of the device. Insert, rotate and tighten the connector into the output. Solenoids must be placed on the area to be treated (they can be applied on the top of stockings or cloths) and can be fixed with the elastic straps. The plain side on the solenoids must be placed in contact with the area to be treated.

#### 4.2 - WELLMAG operating with band EasyCare

WELLMAG offers magnetotherapy treatments at low and high frequency. For high frequency (program 31H), it is mandatory to use WELLMAG with Easycare band or with any optional accessories for high frequency. Solenoids CANNOT be used. For the other low frequency programs (from P01 to P30), solenoids can be used.

#### 4.3 - Power source

You can use WELLMAG with battery or with power supply (230V), connecting the power supply to the device and to the AC mains.

#### 4.4 - Turning on the device

Press key (2) - **ON/OFF**.

#### 4.5 - Program selection

When turning on the device, the section programs (PROG) is displayed. WELLMAG offers:

- **30 preset LOW FREQUENCY programs** (from N.1 to N.30), identified by letter P. These can only be used with solenoids or low frequency accessories;
- **5 adjustable programs** identified by letter U. You can save here programs in low frequency, prescribed by your doctor.;
- **1 HIGH FREQUENCY program (N.31 HF)**, that has an antalgic effect thanks to its automatic scanning mode. This program can be used with the Easycare band or with high therapy accessories (Cap14);

To select the desired program, use the keys (+)/(-).

Programs Table		
Programmi	Nº	DURATA (min)
<b>LOW FREQUENCY</b>		
<i>Fractures</i>	1	75
<i>Osteoporosis</i>	2	90
<i>Joints</i>	3	60
<i>Edemas</i>	4	40
<i>Strains</i>	5	30
<i>Tendonitis</i>	6	45
<i>Lumbago</i>	7	60
<i>Myalgia</i>	8	60
<i>Acute pain</i>	9	60
<i>Chronic pain</i>	10	90
<i>Arthritis</i>	11	35
<i>Periarthritis</i>	12	25
<i>Arthropathies</i>	13	45
<i>Small joints osteoarthritis</i>	14	35
<i>Large joints osteoarthritis</i>	15	50
<i>Coxoarthrosis</i>	16	40
<i>Dislocation</i>	17	60
<i>Widespread contractures</i>	18	60
<i>Contracture</i>	19	40
<i>Myositis</i>	20	60
<i>Bursitis</i>	21	40
<i>Acute epicondylitis</i>	22	40
<i>Foot tendonitis</i>	23	30
<i>Cervical pain</i>	24	50
<i>Sciatalgia</i>	25	60
<i>Acne</i>	26	30
<i>Cellulitis</i>	27	30
<i>Wrinkles</i>	28	30

<i>Couperose</i>	29	30
<i>Stretch marks</i>	30	30
<b>HIGH FREQUENCY</b>		
<i>HF</i>	31	30

#### 4.6 - Start of the therapy

When the desired program appears on the display, press START. Once pressed, the section TIME is displayed, where you can see the remaining time.

**NOTE:** if you press START and the solenoid is not inserted in the instrument, the stimulation stops, the device makes a sound and the display shows the message "SOL"; insert the connector and press 2 START.

#### 4.7 - Setting the frequency in preset low frequency programs

Frequency (Hz) is preset but can be adjusted manually from [1-200]Hz with interval of 1Hz. Go to FREQ menu using the key ↑ and change the value using (+)/(-) keys. To go back to TIME, press ↓.

#### 4.8 - Setting the intensity in preset low frequency programs

Intensity (G) is preset but can be adjusted manually from [1-100] Gauss with interval of 1G. Go to INTEN menu pressing twice the key ↑ and change the value using (+)/(-) keys. To go back to TIME, press ↓.

#### 4.9 - Setting Time

It is possible to increase a program duration (Timer) to perform a longer treatment (few hours or night time, up to 8 hours maximum). This can be done in low and high frequency programs. To set the time while performing the treatment, use the keys ↑ and ↓ to go to the TIME menu, then change the value with (+)/(-). Each pressing increases or decreases time by 5 minutes.

**NOTE:** we recommend using the power supply if the device is used for 2-3 hours sessions.

**PRECAUTION:** for the duration of the session, please always refer to the doctor that prescribed you the magnetotherapy, especially for those of 4-8 hours. We recommend not to change the values of the preset programs, unless specified by the doctor.

**NOTE:** if values are changed while performing the treatment, those will not be saved. So to use them again during the following sessions, you will have to change them again.

#### 4.10 - End or interruption of the session

If you want to stop the device before the program ends, press the key (2) **PAUSE** once for a break (to continue press (2) **START**); hold pressing (2) PAUSE to end the therapy (the device will turn off)

#### 4.11 - Turning off the device

To turn off the instrument press and hold the key (2) **ON/OFF** for some seconds. After the end of a program, if the device is not used again, it automatically turns off after a few minutes.

#### 4.12 - Symbols on display

MAINS FEEDING: power supply through electrical power (charger connected).

Connected solenoid

BATTERY feeding with charge level

#### 4.13 - Verification of the solenoids operation

Magnetotherapy cannot be detected, therefore a magnet is provided with the device, in order to check its functioning. Please proceed to the test below:

1. **Connect the solenoids cable to the device**
2. **Hold the magnet between fingers to put it on the top of the solenoid**
3. **Turn on the device and start program P10**
4. **If the device is working properly, you will notice the magnet vibrating between fingers. This is due to the continuous polarity inversion on the solenoids.**  
If you want to feel a stronger vibration, increase intensity of P10.
5. **Run the same test on the other solenoid**

#### NOTE:

Run the test with P10 program as this is where the vibration is felt the most. If working with this program, it will also work with the other programs.

#### NOTE:

the magnet is only working when using solenoids and low frequency programs. It cannot be used with Easycare band (magnet cannot detect high frequency)

#### 5.1 - Setting of free program

To access to the setting from PROG menu area use the keys (+)/(-) and go to the U programs. Use the keys  $\uparrow$  and  $\downarrow$  to go to INTEN, FREQ, TIME menu. Change the parameters as desired with the keys (+)/(-). Once the parameters have been selected, press key 2 START to start the selected therapy. When the device will be turned off the program will be saved.

NAME ON DISPLAY	PARAMETER	AVAILABLE VALUES
Freq.	Frequency of the impulse	1-200 step 1 Hz
Inten	Intensity	1-100G
Time	Program Duration	5 minutes to 8 hours

To help you choosing the setting in the free programs, please check the guidelines below: in the first column you find the available parameters with the corresponding values, while in the second column you find the corresponding type of application.

**NOTE:** if, after having chosen a program, you want to change to another one, you need to turn off the device with key (2) OFF and turn on again with key (2) ON

Frequency (Freq.)	Effect
1-30 Hz	Warming up/Relaxing/Antistress*
30-60 Hz	Vascular/Altered functions*
60-80 Hz	Analgesic
80-100 Hz	Regenerating
Program Duration (Time)	Goal
Up to 30 min	Analgesic / relaxing effect
30-60 min	Regenerating effect
More than 60min	Bone consolidation/Osteoporosis/Pseudoarthrosis

Analgesic or regenerating effect in long applications (2-4 hours).

See the following examples of free programs for most common applications:

ANALGESIC/ANTINFLAMMATORY	REGENERATING
Freq. - 80	Freq. - 100
Inten - 80	Inten - 100
Time - 45	Time - 60

## 6.1 - Therapy planning

**Always perform the magnetotherapy following your medical prescription.** You can perform the therapy every day, following your medical prescription, until improvement is reached. You can perform the therapy two or three times per day, with 2-3 hours of break between sessions, and maximum of 8 hrs. **If the problem persists, please consult your doctor.**

## 6.2 - Position to be maintained during the sessions

Maintain a relaxed position during each session. Depending on the area of application, the body is lying supine or prone. The position must be maintained during the whole session to facilitate the effects produced by the magnetic field.

## EXAMPLES OF APPLICATIONS WITH SOLENOIDS

The **Easycare band** can be used open on the area to treat or can be wrapped on the area.

Solenoids can be used in pair or individually (in case the area to treat is a small one). They can be used one facing the other, to intensify the effect on the area) or one next to the other (Fig.10) as for example in case of lumbago and cervical pain..

**NOTE:** the intensity of the program is calculated using both solenoids. Of only one solenoid is used, the intensity will be cut by the half (e.g. P.1 Fracture, intensity 96G with both solenoids, 48G if only one is used)

Fig.1 - Cervical pain

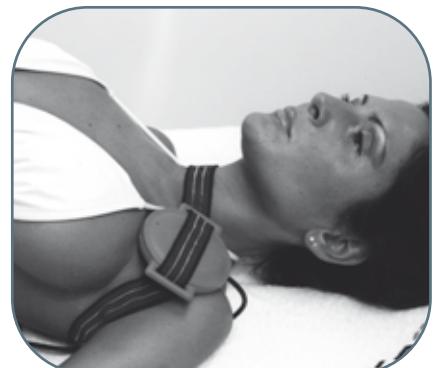
Fig.2 - Shoulder pain front area

Fig.3 - Shoulder pain back area

Fig.4 - Sterno

Fig.5 - Ribs

Fig.6 - Elbow

Fig.11 - Back, upper part

Fig.12 - Back, lower part

Fig.7 - Wrist

Fig.8 - Hand

Fig.13 - Back + Femur

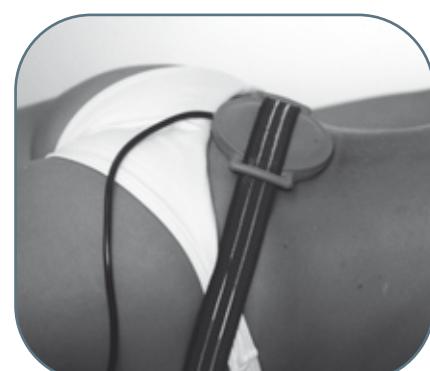
Fig.14 - Sacrum area

Fig.9 - Fingers

Fig.10 - Hips

Fig.15 - Hips - side

Fig.16 - Knee - side

Fig.17 - Knee - front, back

Fig.18 - Tibia and fibula

Fig.19 - Ankle

Fig.20 - Foot

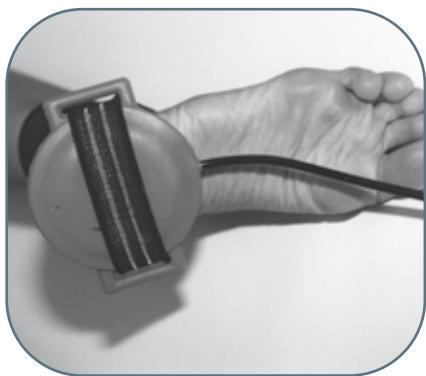
Fig.21 - Hand

WELLMAG can be powered from mains with the endowed power supply or from the internal rechargeable battery Li-ion 12V - 3,8Ah, with an autonomy of some hours under normal use conditions. The guaranteed autonomy may change depending on the program, intensity and time of application.

For the correct identification of battery and power supply, consult the Chap. "Technical features". Neither the power supply nor the battery must be replaced from not experienced staff. Do not use spare parts which are not provided or approved by the manufacturer.

### 7.1 - Charge indication

When WELLMAG is powered by the inside battery on the bar of the symbols of the display the symbol Battery it appears "■■■■■"

Icons:

battery charged

empty battery

intermediate levels of charge

When the empty battery icon is displayed, please charge the device. If not, the device will automatically turn off.

### 7.2 - Battery recharge

To recharge the battery of WELLMAG:

- 1) Turn off** the device;
- 2) Plug the power supply** into the device and into mains;
- 3) Leave the device charging** (on the display the battery icon will blink while charging). When full charge is completed (approx. 3 hours), the battery charged icon will appear (Once in a while we recommend leaving the device charging for 8-10 hours)
- 4) Disconnect** the power supply from mains and WELLMAG.

### 7.3 - Battery maintenance

The life of a rechargeable battery depends on the number and type of cycles of charge / discharge. If WELLMAG is not used frequently, we recommend recharging the battery monthly; to prolong the life of the battery we recommend once in a while discharging it completely (see empty battery symbol) and to have a complete 8-12 hours recharge cycle.

### 7.4 - Battery replacement

When the battery's life is significantly lower than it used to be, we recommend replacing it. For this, please refer to an authorized Centre of Assistance that will replace it under all security and safety requirements.

Never reverse the polarity of the connections of the battery, not to cause the destruction of the electronic circuits.

### 7.5 - Precautions for use

- 1) Do not throw the exhausted battery with the ordinary wastes. Please take them to the corresponding collection site.
- 2) Do not manipulate/ throw the battery into a fire.
- 3) Do not short-circuit the terminals.
- 4) Avoid sparks or flames, above or around the battery.
- 5) In case of direct contact with liquid leaking from the battery, wash immediately with water. In case of direct contact with eyes, wash abundantly with water and consult immediately a doctor.

Error message on display	Problem	Solution
<b>SOL</b>	Unproper connection of the solenoids into the device	Make sure the connector is properly inserted to start the therapy. If the problem persists, contact the Technical assistance
<b>OL</b>	Over Load, problem in the solenoids	Contact the Centre of Assistance

### 9.1 - Solenoids

The connection's cable must be periodically revised to check the possible presence of cracks, possible cause of dispersion of the magnetic fields. Clean the solenoids periodically with a damp cloth.

### 9.2 - Device and power supply

We recommended cleaning the device and power supply with a damp cloth. **Do not use in any liquid, because these devices are not liquid proof (IP20).**

### 9.3 - Power supply replacement

Check periodically the level of wear of the isolating parts of the power supply before connecting it. If it's damaged, also only partially, replace immediately the power supply.

### 9.4 - Immediate maintenance

Immediate maintenance must be performed by authorized staff if one of the followings occurs:

- external mechanical stress (eg. serious falls);
- strong overheating (eg. if the device has been left close to sources of intense heat);
- some liquids may have been penetrating the device;
- the power supply, or other parts of the device are damaged, broken or missing;
- the functionality of the instrument seems altered.

Use the device with the endowed accessories only

Run every year a secur-tester to check the device following the legislation EN60601-1. The useful life of the device is guaranteed only if such maintenance is regularly performed.

- !** Pay attention and use carefully the device with solenoids, not to compromise the effectiveness of the treatment.
- !** Make sure that the electric system is in conformity with the actual safety norms.
- !** The instrument has degree of protection IP20 (see Chap. "Technical features"). Do not use it close to liquids, because it's not liquid proof.
- !** Do not use close to cellular telephones (maintain at least one meter of distance).
- !** The stability of the emission can be affected when operating close (less than 1 meter) to an instrument with short waves therapy or microwaves.
- !** Do not use with the presence of devices to monitor the patient's vitals.
- !** Do not connect simultaneously the patient with the WELLMAG and with a surgical instrument High Frequency, to avoid dangers for the patient and for the instrument.
- !** The device works accordingly to its specifications if the environment is maintained to a temperature between the 5° and the 40° C and with humidity inferior to the 80%. The same conditions must be maintained during transportation and the storage.
- !** In case of malfunctioning, do not use and request service of reparation.
- !** Do not operate in proximity of inflammable substances.
- !** Use only gels and accessory that are endowed with the device.
- !** It's important to inform the patient on the type of sensation they will experience during the therapy. In case the patient perceive a different sensation, it is important to stop the treatment and to remove the solenoids.
- !** If the intensity is causing overheating or pain in the treated zone, it is necessary to reduce immediately the intensity of stimulation or eventually interrupt the treatment.
- !** Keep away from children.

**11.1 - Power supply**


Power supply: Biron brand  
 PRI: 230V~ 50Hz SEC: 13,6V- 1,2A  
 Internal power source: Rechargeable battery Li-Ion 12V-3,8 Ah

**11.2 - Output features**

Max intensity of magnetic induction (B): 100 Gauss  
 Frequency (f): 1-200 Hz

**11.3 - Other features**

- **Dimensions:** 140x105x40h [mm]
- **Weight:** 400 [g]
- **Class: II Type:** BF
- **Classification to liquids entry:** IP20
- **Safety in presence of flammable anaesthetic gas:** it's not of AP or APG category
- **Type of operation:** continuous

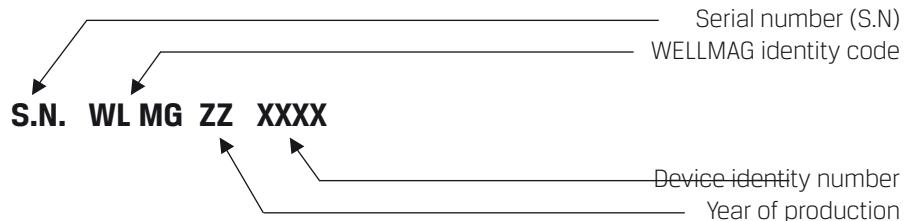BF TYPE DEVICE

ATTENTION, CONSULT THE ATTACHED DOCUMENTATION

THIS DEVICE IS MARKED ACCORDING TO DIRECTIVE  
CEE 93/42.WASTE DISPOSAL FOLLOWING THE CEE DIRECTIVE 2002/96  
AND 2003/108

- C. Menarini, M. Menarini: ***Manuale di terapia fisica***, Aulo Gaggi Editore, Bologna 1985
- M. Moselli, M. Manca: ***Fisioterapia pratica***, Ed. Minerva Medica, Torino 1993
- B. Gialanella, G. D'alessandro, R. Santoro: ***Terapia fisica pratica***, ED. Marrapese, Roma 1997
- Vasta: ***Manuale pratico illustrato di terapia fisica***, ED. Marrapese, Roma 1998
- Cisari, G. Severini: ***Fisioterapia clinica pratica***, Edi-ermes, Milano 1999
- T.Thorossian: ***Magnetic field therapy***, Ed. NeoMedica, Vienna 1999
- G. Nanni, G. S. Roi, D. Vasapollo: ***Le lesioni muscolari dell'arto inferiore nello sportivo***, ED. Marrapese, Roma 2000

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