CE

HSD

---

<b>ITALIANO</b>	<b>Programmi per l'uso estetico &amp; benessere</b>	<b>PAGE 3-5</b>
<b>ENGLISH</b>	<b>Programs for Aesthetic &amp; Wellness use</b>	<b>PAGE 8-10</b>
<b>FRANÇAIS</b>	<b>Programmes à usage esthétique &amp; bien-être</b>	<b>PAGE 11-14</b>
<b>ESPAÑOL</b>	<b>Programas de uso estético &amp; bienestar</b>	<b>PAGE 15-18</b>
<b>DEUTSCH</b>	<b>Programme für Beauty &amp; Wellness</b>	<b>PAGE 19-22</b>

# ----- ITALIANO -----

## Programmi Estetici-Benessere

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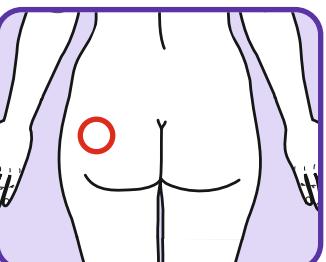
----- ENGLISH -----

## Programs for Aesthetic-Wellness use

 **Information**

*In programs where the area of the body to be treated allows it, the use of the ultrasonic handpiece can be replaced with double-head diffusers.*

## PROGRAMS

Application notes to use the device correctly, knowing which program to choose and the action it performs, thus fighting your imperfection as effectively as possible.

**Bump**

The ultrasound irradiation in this action causes a massage of considerable intensity, which by means of the oscillations penetrates deeper into the epidermis.

**ACTION** WHEN It should be used in all those advanced stages of localized cellulite, especially in the case of soft or flaccid cellulite.

**Modeling**

**ACTION** This therapy is widely used in cases of localized lipodystrophy by breaking the fibrotic tissue present which hinders both blood and lymphatic circulation.

**WHEN** Benefiting from its characteristic in shaping and stimulating it is particularly indicated in the case of hard or solid cellulite.

**Maintenance**

**ACTION** Its stimulating effect on local blood circulation improves skin turgor and cellular oxygenation, releasing excess fluids and toxins from the subcutaneous connective tissue.

**WHEN** It is indicated as maintaining the cycle of sessions carried out, in order not to lose the results achieved.

**Facial scrub**

**ACTION** It reactivates the circulation and promotes cell turnover: the skin is purified in depth and prepared to better absorb the active ingredients of moisturizing and anti-wrinkle creams. The exfoliating action also has beneficial effects on blackheads which are reduced or even disappear with ultrasonic waves. The treatment should be performed once a day.

**Face massage**

**ACTION** It increases the elasticity and tone of the tissues by reducing the possible formation of post-acne scars and increases the effect of anti-acne products. The treatment should be performed once a day.

**Smoothing for the face**

**ACTION** It smoothes and relaxes (expression) wrinkles and if used preventively it can delay skin aging or prevent it from worsening. Thanks to its thermal effects, ultrasound heats the tissues and promotes biochemical and metabolic processes. The treatment should be performed once a day.

**Smoothing**

It recalls the clean plasma throughout the thickness of the dermis and restores the right physiological dimension to the skin connective tissue, improving both the elasticity and the superficial irregularity of the epidermis.

**ACTION** **WHEN** It is indicated in all those cellulite blemishes, in which the skin surface has irregularities, such as the classic "Orange peel skin" and "Rough Skin".

**Draining**

**ACTION** Its beneficial action facilitates the drainage of the lymphatic connectors and of the respective attached glandular stations, facilitating the reabsorption of liquids.

**WHEN** Benefiting from its draining action, it is recommended in the case of edematous cellulitis, as liquids are the main cause, also recommended against orange peel skin.

**Reactivating**

**ACTION** Stimulates and reactivates blood and lymphatic circulation for skin improvement, reactivating cell metabolism, favoring the exchange of liquids between cells.

**WHEN** It is especially indicated in the initial stages of cellulite, as it stimulates and reactivates the dormant metabolism.

**Carrier**

It favors the penetration of creams, gels or other products into the biological tissue, favoring their absorption and therefore their effect, also indicated for depilatory treatments.

## PROGRAM

## OUTER ARMS

ACTION	
SHOCK	01
MODELING	02
MAINTENANCE	03

## INNER ARMS

ACTION	
SHOCK	04
MODELING	05
MAINTENANCE	06

## HIPS AND BACK

ACTION	
SHOCK	07
MODELING	08
MAINTENANCE	09

## ABDOMIN. AREA

ACTION	
SHOCK	10
MODELING	11
MAINTENANCE	12

PROGRAM	
<b>CULOTT. CHEVAL</b>	
ACTION	
SHOCK	13
MODELING	14
MAINTENANCE	15

PROGRAM	
<b>GLUTEUS MUSCLE</b>	
ACTION	
SHOCK	16
MODELING	17
MAINTENANCE	18

PROGRAM	
<b>OUTER THIGHS</b>	
ACTION	
SHOCK	19
MODELING	20
MAINTENANCE	21

PROGRAM	
<b>INNER THIGHS</b>	
ACTION	
SHOCK	22
MODELING	23
MAINTENANCE	24

PROGRAM	
<b>KNEES</b>	
ACTION	
SHOCK	25
MODELING	26
MAINTENANCE	27

PROGRAM	
<b>INNER KNEE</b>	
ACTION	
SHOCK	28
MODELING	29
MAINTENANCE	30

PROGRAM	
<b>CALVES</b>	
ACTION	
SHOCK	31
MODELING	32
MAINTENANCE	33

PROGRAM	
<b>CELLULITE</b>	
ACTION	
DRAINING	34
SMOOTHING	35
MAINTENANCE	36

PROGRAM	
<b>ROUGH SKIN</b>	
ACTION	
REACTIVATING	37
SMOOTHING	38
MAINTENANCE	39

PROGRAM	
<b>MAN</b>	
ACTION	
SHOCK	40
MODELING	41
MAINTENANCE	42

PROGRAM	
<b>VEHICULATING</b>	
ACTION	
HIGH	43
MEDIUM	44
LOW	45

PROGRAM	
<b>SCALP</b>	
ACTION	
STIMULATING	49
CLEANING	50
ACTIVATING	51

PROGRAM	
<b>IMMERSION</b>	
ACTION	
1 MHz	52
3 MHz	53

#### PROBE TESTING


2-minute program to be used only and exclusively to check the correct functioning of the treatment handpiece

PROGRAM 114


**PATHOLOGY**
**ACNE AND SPOTS**


FREQUENCY	3 MHz
POWER	0,5 - 1 W/cm <sup>2</sup>
EMISSION	puls. 75%
DURATION max	12 minutes
PROGRAM	99
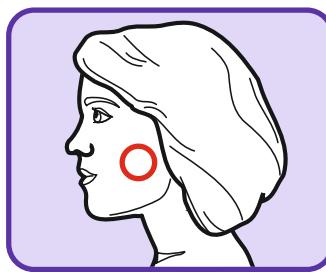
**PATHOLOGY**
**CELLULITE  
UPPER LIMBS**


FREQUENCY	1 - 3 MHz
POWER	1,0 - 1,5 W/cm <sup>2</sup>
EMISSION	puls. 50%
DURATION max	12 minutes
PROGRAM	100

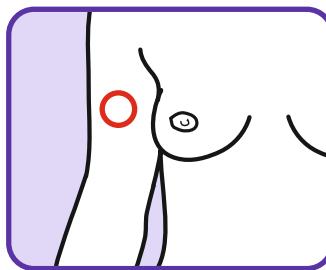**PATHOLOGY**
**CELLULITE LOWER LIMBS**


FREQUENCY	1 - 3 MHz
POWER	1,5 - 2 W/cm <sup>2</sup>
EMISSION	puls. 75%
DURATION max	12 minutes
PROGRAM	101

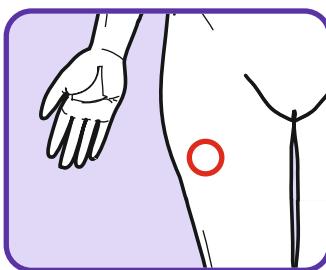**PATHOLOGY**
**CELLULITE ABDOMEN**

FREQUENCY	1 MHz
POWER	2 - 2,5 W/cm <sup>2</sup>
EMISSION	puls. 50 - 75%
DURATION max	12 minutes
PROGRAM	102

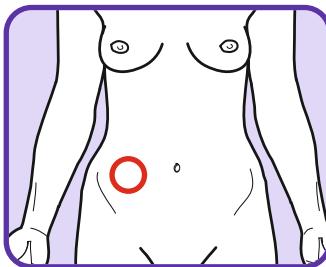**PATHOLOGY**
**CELLULITE BUTTOCKS**

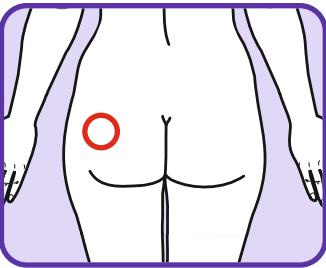FREQUENCY	1 MHz
POWER	2 - 2,5 W/cm <sup>2</sup>
EMISSION	puls. 75 - 90%
DURATION max	12 minutes
PROGRAM	103

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