*Well
Store*®

Terapeutico

HSD

CE

1370

ITA - ENG - FRA

ITALIANO	Manuale d'uso e programmi per l'uso terapeutico User	PAGE 3-27
ENGLISH	manual and programs for therapeutic use	PAGE 28-51
FRANÇAIS	Manuel d'utilisation et programmes à usage thérapeutique	PAGE 52-75

ITALIANO

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ENGLISH

Dear Customer,
we congratulate you on your choice. It now has at its disposal a high performance product, built according to the most recent and strict European regulations, and able to guarantee a high standard of quality and safety. Please pay close attention to this manual by reading it in its entirety in order to familiarize yourself with the appliance.

HSD S.r.l.

INDEX	
COMPONENTS DESCRIPTION	30
WARRANTY	33
GENERAL SAFETY INSTRUCTIONS	33
REFERENCE STANDARDS	34
DECLARATION OF CONFORMITY	34
CONTRAINDICATIONS	35
TECHNICAL AND FUNCTIONAL FEATURES	36
TECHNICAL DATA	36
IDENTIFICATION	37
PUT IN ACTION	38
MAINTENANCE	41
ACCESSORIES AND SPARE PARTS	42
MAGNETOTHERAPY LF, CMP-HF, PHOTOCATALYSIS, BLUE AND INFRARED PROGRAMS	43
QUESTIONS AND ANSWERS	51
TROUBLESHOOTING	51

MANUFACTURER:
HSD S.R.L.
STRADA ACQUASALATA, 11 - INT. 6 - 47899
SERRAVALLE RSM

T. (+378) 0549.941496 - F. (+378) 0549.976456
www.hsd.sm - info@hsd.sm

No part of this manual may be reproduced, copied or disclosed by any means without the written authorization of HSD S.r.l. Any infringements will be prosecuted in the manner and within the times provided for by the law.

All names and trademarks mentioned in this manual are the property of their respective manufacturers.

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INTRODUCTION


This technical document describes the rules for the correct use and user maintenance of the appliance.

Compliance with the rules and instructions given here will allow you to use it as safely as possible.

SYMBOLS USED	
	ATTENTION Calls attention to situations or problems that could affect safety.
	INFORMATION Calls attention to important situations of a general nature.

INFORMATION

If, after reading, doubts or uncertainties persist about the operations to be performed, it is necessary to contact the trusted dealer or the manufacturer.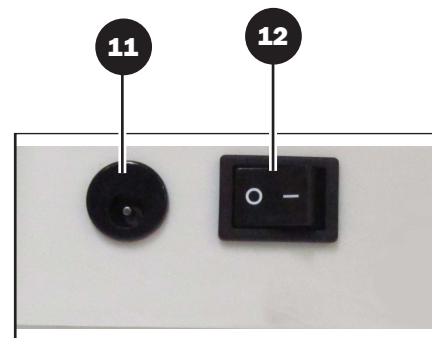

BACK

DESCRIPTION OF THE APPLIANCE AND FUNCTIONS

- 1 DISPLAY**
used for dialogue with the operator

- 2 LED ON**
indicates that the appliance is connected to the mains

- 3 LED STOP**
indicates that the appliance is not in operation

- 4 LED PAUSE**
indicates that the appliance is in operation but temporarily stopped

- 5 LED ACTIVE**
indicates that the appliance is in operation

- 6 PROGRAMS + & - BUTTON**
allows you to select forward and backward programs in succession

- 7 START / STOP LF BUTTON**
for starting, pausing and stopping the MAGNETO LF treatment

- 8 START / STOP HF BUTTON**
for starting, pausing and stopping the MAGNETO HF treatment

- 9 MAGNETO LF treatment speaker connector**

- 10 MAGNETO HF treatment diffuser connector**


- 11 Power connector housing**

- 12 General ignition switch**

CONTENTS OF THE PACKAGE

- Device 220 ÷ 240 VAC 50 - 60 Hz
- 1 kits with two elastic bands (60 cm and 100 cm)
- Cable for accessories
- Alimentation cable
- User manual with treatment guide
- Transport bag
- Optional accessories depending on the configuration

The device is supplied with all the equipment indicated above, so once the packaging has been opened, check that the equipment is complete. If any item is missing, please contact the seller.

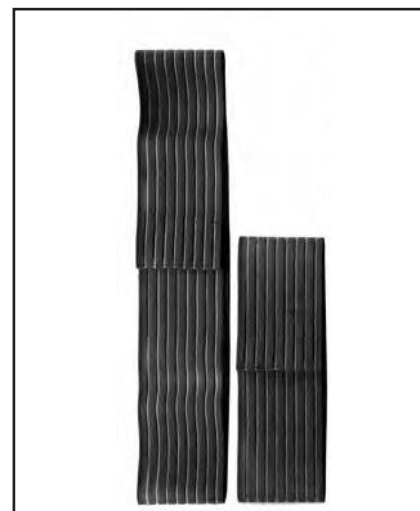BANDS KIT

OPTIONAL ACCESSORIES

HF MAGNET DIFFUSER

LF MAGNET DIFFUSER

MAGNETO - LIGHT DIFFUSER

MAGNETO - LIGHT HANDFUL

MAF DETECTOR

MATTRESS MAF

MAGNETIC VEST


USE OF ACCESSORIES

• MAGNET HANDPIECE / DIFFUSER - LIGHT


Useful accessories for those very annoying forms of acute / chronic pain; due to the strong concentration of electromagnetic energy on the handpiece, a maximum time of 15 minutes for 20 sessions is recommended. These accessories can be used in any program instead of the speakers normally used.




• **CERVICAL BAND**
Accessory useful for neck and cervical treatment.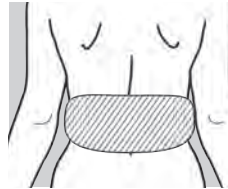

• **LUMBAR FASCIA**
Useful accessory for the treatment of the lumbar area.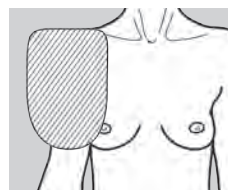

• **SHOULDER KNEE / ELBOW BAND**
Multifunctional accessory useful for the treatment of different areas of the body.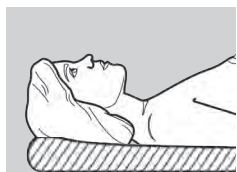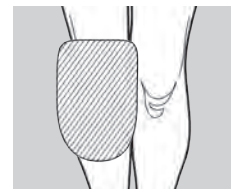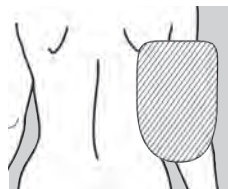

• MATTRESS MAF

The mat is usually placed under the sheet of the bed or on an armchair to allow a comfortable

therapy for those who need a more prolonged action.


• MAGNETIC VEST

This accessory is usually used for those suffering from trunk problems (pelvis, back, neck, etc.).

• FACIAL BAND

A useful accessory for those suffering from neuralgia (trigeminal, den taints, etc.), but also for those who want to delay the aging processes (anti - aging).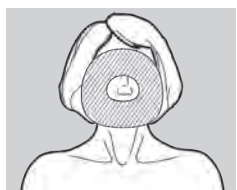

• MAF DETECTOR

It is a device for verifying the correct functioning of high frequency electromagnetic fields generators for magnetotherapy. It is used by probing the accessory (mattress, band ...) connected to the MAF device in operation.

SHOULDER / KNEE / ELBOW BAND

CERVICAL BAND

FACIAL BAND

LUMBAR BAND

WARRANTY

Each appliance is subject to careful checks and is tested before being put on sale. The consumer is the owner of the rights specified by current legislation. The warranty has a duration of two years from the date of purchase and is extended to all parts, except those subject to wear. It lapses if the HSD device is not used properly or tampered with by unauthorized personnel, or if the defect is not reported within two months from the date on which it appeared. The parts deemed by HSD S.r.l. defective at the time of manufacture will be replaced free of charge.

HSD S.r.l.

CAUTION!

The warranty also lapses in the event of tampering or removal of the sticker placed under the casing of the device.

USEFUL LIFE

HSD S.r.l. provides that the duration of maintenance of the functionality and safety of the device in the conditions of use, maintenance care indicated in the User Manual is 6 years. The devices are guaranteed, with correct use and maintenance, by HSD S.r.l. for 24 months.

HSD S.r.l.

GENERAL SAFETY INSTRUCTIONS

Some general safety guidelines are summarized below:

- *Carefully read the instructions for use before starting the appliance;*
- *do not use the device in the presence of flammable mixtures with air or oxygen or nitrous oxide, absolutely avoid explosive environments and humid environments or environments in which large quantities of water are present, as the apparatus is not protected against the abundant penetration of liquids.*
- *check, before each treatment, the integrity of the power cable, the cable of the treatment speakers and connectors;*
- *to power the device, use only the supplied three-pole cable or request it from HSD S.r.l. ;*
- *never carry out any operation on the appliance with damp or wet hands;*
- *do not immerse the appliance in water for cleaning;*
- *consult the relevant paragraph*

the programs available before using the appliance;

- *do not place weights, and in particular containers with liquid on the appliance, do not spill liquids on it;*
- *in the event of abnormal behavior or malfunction, stop the treatment and turn off the device, do not try to open or repair the device. Please contact HSD S.r.l. for any type of intervention or control.*

WEEE: This product complies with the Directive 2012/19 / EU.

The crossed out trash can symbol shown on the appliance indicates that the product, at the end of its useful life, having to be treated separately from household waste, must be disposed of in a separate collection center for electrical and electronic equipment or returned to the retailer when purchasing a new equivalent appliance. the user is responsible for transferring the appliance at the end of its life to the appropriate collection facilities, under penalty of the penalties provided for by current waste legislation. Adequate separate collection for the subsequent start-up of the decommissioned appliance for recycling, treatment and environmentally compatible disposal helps to avoid possible negative effects on the environment and health and favors the recycling of the materials of which the product is made. For more detailed information regarding the collection systems available, contact the local waste disposal service. Manufacturers and importers fulfill their responsibility for environmentally compatible recycling, treatment and disposal both directly and by participating in a collective scheme.

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CONTRAINDICATIONS

⚠ CAUTION!

The device is locally contraindicated:

- in the pregnant woman
- in cardiac pacemakers;
- on tumor areas

The device is not recommended:

- in the severely debilitated patient who is unable to communicate;

⚠ CAUTION!

In case of doubts or concerns it is necessary to consult a doctor.

INTENDED USE AND APPLICATIONS

CMPs have the following characteristics:

LF frequency: 10 ÷ 100 Hz
 LF flux density: 150 Gauss pp
 CMP LF Radiated: 20 ÷ 40 Gauss
 HF Frequency: 500 ÷ 10000 Hz
 HF Carrier Frequency: 20 MHz

The device is studied and designed to be used in environments such as:

1. Outpatient
2. Physiotherapeutic
3. Aesthetic
4. Domestic

The use of this device is allowed to medical personnel, paramedics and to the patient himself if properly informed of the methods used by the device. In case of doubts or concerns, it is necessary to consult a doctor.

- The device is studied and designed to be used in environments such as:

1. Outpatient
2. Physiotherapeutic
3. Aesthetic
4. Domestic

- The use of this device is allowed to medical personnel, paramedics and the patient himself if properly informed of the methodologies used by the device. In case of doubts or concerns, it is necessary to consult a doctor.

Specific therapeutic applications

- Pain reliever; - Arthritis; - Lumbosacral arthrosis; - Asthenia; - Muscle atrophy;
- Bursitis; - Brachialgia; - Cervical;
- Bruises; - Sprains; - Pain in the fingers;
- Wrist pain; - Intercostal pains;
- Low intercostal pains; - Jaw pains;
- Flat foot pains; - Vertebral pains;
- Back pain; - Epicondylitis; - Nasal septal fracture; - Bone fractures; - Acute Gout
- Ischialgia; - Metatarsalgia; - Dupuytren's disease; - Osteoporosis; - Rheumatism;
- Sacralgia; - Sciatica; - Spondilo - Arthritis;
- Muscle tears; - Tallonite; - Tendinitis;
- Stiff neck

Specific fitness and beauty applications

- Absorption of active substances on the face and body;
- treatment of superficial wrinkles and sensitive skin;
- acne treatment; - boils on the face and body;
- anti-cellulite treatments

⚠ CAUTION!

The device must not be used for:

1. electrodiagnosis
2. experimental use;
3. operations not included in the examples of use given in the instructions for use and maintenance. Any use other than those described or the use of products that do not comply with HSD specifications is to be considered improper.

Technical and functional characteristics

Manufacturer: HSD S.r.l.
Strada Acquasalata, 11 - int. 6 - 47899
Serravalle RSM T. (+378) 0549.941496
- F. (+378) 0549.976456

TECHNICAL AND FUNCTIONAL FEATURES

MANUFACTURER:

HSD S.r.l.

**Strada Acquasalata, 11 - int. 6
47899 Serravalle RSM**

T. (+378) 0549.941496

F. (+378) 0549.976456

ENVIRONMENTAL CONDITIONS ALLOWED FOR THE OPERATION

TEMPERATURE [T]
+10 ÷ +40 C°

HUMIDITY [RH]
30 ÷ 75%

PRESSURE
ATMOSPHERIC [mbar]
from 700 to 1060 mbar

ENVIRONMENTAL CONDITIONS ALLOWED FOR TRANSPORT AND STORAGE

For the transport and storage of this appliance, the following environmental conditions are recommended:

TEMPERATURE [T] **0 ÷ 40 C°**

HUMIDITY [RH] **25 ÷ 75%**

TECHNICAL DATA

POWER SUPPLY	220 ÷ 240 VAC
POWER FREQUENCY	50 - 60 Hz
ABSORBED POWER	50 W
ELECTRICAL INSULATION CLASS	II
TYPES OF APPLIED PARTS	BF
MAGNET WORKING FREQUENCIES LF	1 ÷ 100 Hz
FLOW DENSITY LF MAX.	200 GAUSS pp
CMP LF RADIATED	1 ÷ 100 GAUSS
WORKING FREQUENCIES OF HF MAGNET (CARRIER)	20 MHz
HF MAGNET WORKING FREQUENCIES	500 ÷ 10000 Hz
FLOW DENSITY HF MAX.	7 mW (ON RESISTIVE LOAD 75 OHM)
CMP HF RADIATED	3 mW (ON RESISTIVE LOAD 75 OHM)
PHOTOCATALYSIS WORK FREQUENCY (LIGHT)	1 ÷ 100 Hz
WAVELENGTH	420 ÷ 920 nm/P
THERAPEUTIC PROGRAMS	92
PROGRAMS DURATION	45 minutes

DIMENSIONS AND WEIGHT

HEIGHT	50 mm
DEPTH	200 mm
WIDTH	160 mm
WEIGHT	550 g

IDENTIFICATION

On the back of the device and on the ultrasound head, identification plates are affixed with the technical data to be communicated to the Authorized Service Center, to the Manufacturer

or to the Reseller in case of request for information or technical assistance.

- A** *Maker*
- B** *CE marking*
- C** *Attention, consult the attached documentation*
- D** *Physiological effects: high temperature*
- E** *BF Type of applied parts*
- F** *Class II device*
- G** *Consult instructions for use*
- H** *Technical data*
- I** *Model, serial number and year*
- L** *Waste disposal: treat separately from household waste*

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Mod: MANIPOLO ML UNIVERSALE
SN 000000 / 00

INFORMATION

Damage, removal or alteration of the identification plate must be immediately reported to the Manufacturer, the Authorized Service Center or the Dealer. Warranty interventions will not be possible if the identification plate is damaged, removed or altered.

PUT IN ACTION

Remove the appliance from the packaging and place it on a stable horizontal rigid surface, suitable for supporting its weight, near a power outlet. The space around the appliance must not be less than 20 centimeters on all sides.

CAUTION!

The installation of the device is prohibited:

- in humid environments;
- near large quantities of water;
- near heat sources;
- in direct sunlight.

CAUTION!

Never connect and disconnect the treatment diffusers when the device is switched on.

CONNECTION OF THE TREATMENT HEAD

Insert the speaker connector into the sockets provided on the front panel paying attention to the insertion guide.

It is forbidden to use the device in atmospheres that may contain flammable and / or anesthetic gases.

CONNECTION TO THE ELECTRICITY NETWORK

Do not use multiple plugs or power adapters.

Connect the appliance to a tested and efficient socket, equipped with earth. If in doubt, do not connect the device.

Check that the voltage of the power supply corresponds to the voltage and frequency indicated on the appliance plate

(an incorrect power supply voltage can damage it without

consequences for the user). Using the appliance that is not properly connected to the mains can be dangerous. Before inserting the cable


plug into the socket

power supply make sure that the power switch is positioned on

"0" (zero). Insert the power cable in housing "11" near the switch "12", making sure that it is fully inserted. Insert the plug into the socket.

CAUTION!

To avoid strangulation and suffocation with the power cord: mount the unit and the power cord away from the face and out of the reach of children.

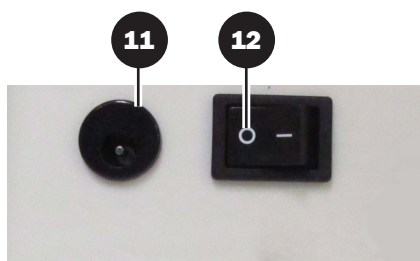METHOD OF USE

The device can simultaneously manage 2 low frequency magnetic diffusers,

2 High Frequency magnetic speakers and 1 additional output High Frequency for use with mats or dedicated accessories.

The aforementioned speakers and accessories share the same program and the two Low and High frequency sections can be enabled and disabled separately by using the relative "START / STOP" buttons.

The device supports 92 different programs to intervene on various treatable pathologies.

When switched on, by placing the switch in position "1", the green "ON" LED lights up to confirm the correct electrical connection. The presentation text appears scrolling on the display:

```

MAGNETOTERAPY LFHF
HSD
  
```

Below, on the second line of the display, the version of the software installed in the machine and the issue date are displayed for a few seconds:

```

1.00 HSD 11NOV11
VERSION ISSUING DATE
  
```

At the end of the previous display, an invitation to choose a treatment program appears:

```

CHOOSE
PROGRAM
  
```

At this point, using the "PROGRAMS ▲ / ▼" keys, one of the 92 available programs can be selected. The display shows the program number and name

of the Pathology:

```

GROUP          PROGRAM
PROGRAM      16
CERVICAL
PATHOLOGY NAME
  
```

Once the program has been selected and the speakers / accessories to be used have been connected, I start the Low Frequency section using the "START / STOP LF" key, while the "START / STOP HF" key starts the High Frequency section. In this case, the "ACTIVE" led also lights up to indicate that one or both sections are in operation.

During the ACTIVE phase, the display shows the group, the program of the group and the name of the pathology in the first line and in the second line, alternately, the programmed parameters of the Low Frequency section:

```

PROGRAM      PATHOLOGY
16          CERVICAL
B: 45'      035G      020F
REMAINING    LF        HF
TIME         MAGNET    MAGNET
              INTENSITY  FREQUENCY
  
```

And of the High Frequency:

```

PROGRAM      PATHOLOGY
16          CERVICAL
H: 45'      1250F
REMAINING    HF
TIME         MAGNET
              FREQUENCY
  
```

In this phase the led bar indicates the emitted power relative to the two sections Low and High frequency. The two sections are independent, so I can act on the respective START / STOP buttons. In case a section is in ACTIVE phase, by pressing the relative START / STOP key you can put it in "PAUSE PHASE": the "PAUSE" led starts flashing and the "ACTIVE" led turns off. The Buzzer also signals this condition with continuous close notes.

I will then be able to have a section in ACTIVE PHASE and a section in PAUSE PHASE; during the

displaying the data relating to the section, the ACTIVE and PAUSE LEDs will switch on appropriately. Both during the ACTIVE PHASE and during the PAUSE PHASE, by pressing and holding one of the two START / STOP keys for two seconds, the treatment program is definitively interrupted. The Buzzer will emit three notes close together, the ACTIVE and PAUSE LEDs turn off and the STOP LED lights up. At the same time the display shows:

```

PROGRAM
ENDED
  
```

If, on the other hand, the current treatment ends when the programmed time has elapsed, the Buzzer emits a long note and the display shows:

```

PROGRAM
ENDED
  
```

Once the current program is finished or interrupted, the machine goes back to the "CHOOSE A PROGRAM" point, the STOP led goes out and only the ON led remains lit.

At this point you can select a new treatment, otherwise you can turn off the device by placing the power switch in the "0" state.

⚠ CAUTION!

Handle the treatment diffusers with care. Careless handling can affect their characteristics. Do not knot or twist the connection cable. Do not use near the device and the spray, liquid or solvent treatment diffusers.

**APPLICATION
OF DIFFUSERS**

The application of the magnetic diffusers is very simple. In fact, it is sufficient to position the diffusers on the parts to be treated (our external thighs example), blocking them with the supplied elastic bands, then activate the treatment.

FUNDAMENTAL RULES TO BE FOLLOWED

⚠ CAUTION!

- If it is necessary to make an interruption "Unexpected", remember to "ALWAYS" pause the machine;
- before proceeding with the application it is advisable to identify the part of the body on which to act. For anticellulite treatments we recommend the upright position for a correct evaluation of the area to be treated, because in a sitting or lying position there is a staggering of the silhouette;
- having identified the exact area, select the type of program and the action on the device;
- the actions that the program proposes provide for application times on small areas, so to act on larger areas it will take a longer amount of time, having to repeat the same program several times depending on the extent of the area to be treated.

END OF TREATMENT

At the end of the treatment:

- turn off the appliance by turning the main switch to "0" (zero). If the appliance remains on, its operation is not compromised;
- disconnect the treatment diffuser from the appliance panel, taking the connector and pulling upwards perpendicularly, avoiding tearing;
- disconnect the cable plug power supply from the socket;
- disconnect the cable power supply from the back of the device by pulling it outwards with very slight vertical oscillations;
- clean the treatment diffuser, avoiding the use of solvents or abrasives, as indicated in the "Maintenance" chapter;
- collect the cables of the treatment and power speakers in an orderly manner, avoiding knotting or twisting them;
- reposition the device and its accessories in the supplied case;
- store in a dry and safe place, out of reach of children, avoiding accidental falls.

⚠ CAUTION!

At the end of each treatment carry out the maintenance operations described in the "Maintenance" chapter indicated with periodicity "A".

MAINTENANCE

i INFORMATION

For maintenance, only original products and spare parts must be used. Improper use of non-original products and spare parts causes the warranty to void. The information in this chapter is provided in order to allow the user to obtain the best performance from the appliance in the best safety conditions.

MAINTENANCE OPERATION	PERIODICITY	
	M	F
CHECK OF THE INTEGRITY OF THE POWER CABLE	M	F
CHECK OF THE INTEGRITY OF THE TREATMENT DIFFUSER CABLE	M	F
CHECK OF THE INTEGRITY OF THE TREATMENT DIFFUSER	A	A
CLEANING THE APPLIANCE	A. N.	A. N.
CLEANING AND DISINFECTION OF THE TREATMENT DIFFUSER	A	A
PERIODIC CALIBRATION	2 Y	2 Y
ELECTRICAL CHECKS OF THE APPLIANCE	2 Y	2 Y
REGULAR USE		
FREQUENT OR INTENSE USE		
i INFORMATION		
Periodicity:		
A	At each treatment	F Fortnightly
M	Monthly	A. N. As needed
		2 Y Biennial

⚠ CAUTION!

During maintenance of the appliance:

- never carry out any operation with damp or wet hands;
- do not immerse the appliance in water for cleaning, use a cloth soaked in non-abrasive neutral detergent;
- do not try to open or repair the appliance. Contact the Authorized Service Center for any type of intervention or control.

⚠ CAUTION!

The biennial maintenance of the appliance must be carried out exclusively by HSD S.r.l.

CLEANING AND DISINFECTION

⚠ CAUTION!

Before each action, set the ignition switch to "0" (zero) and disconnect the appliance from the power outlet. Handle the treatment diffuser with care. Careless handling can affect its characteristics.

Cleaning the appliance

Remove the dust on the outside of the appliance using a soft cloth moistened with water, or with an aqueous solution consisting of 75% water and 25% denatured ethyl alcohol.

⚠ CAUTION!

It is forbidden to use abrasives or chemicals for cleaning.

Cleaning the diffuser of treatment

Disconnect the treatment diffusers from the device and remove any residual cream or gel using a soft cloth moistened with water, or with an aqueous solution consisting of 75% water and 25% denatured ethyl alcohol. After cleaning, dry well and remove any residual moisture from the treatment diffusers before storage or subsequent use.

⚠ CAUTION!

It is forbidden to use abrasives or chemicals for cleaning. When cleaning, avoid contact between the connector and the cleaning solution used.

Disinfection of treatment diffusers

To disinfect the treatment diffusers, use a cotton ball or gauze, lightly soaked in a non-irritating and alcoholic antiseptic solution (for example the type used for small abrasions or grazes, available in pharmacies).

⚠ CAUTION!

It is forbidden to use chemical or abrasives for disinfection. During disinfection, avoid contact between the connector and the antiseptic solution used. Follow the directions and precautions for use provided by the manufacturer of the antiseptic solution.

Control of treatment diffusers

At each cleaning and - or disinfection of the treatment diffusers and in any case at the end of each treatment it is necessary to check that there are no cracks or fissures that could allow the entry of conductive liquid. If the treatment diffusers show cracks or cracks, replace them immediately.

ACCESSORIES AND SPARE PARTS

For any orders of accessories spare parts, consult the accessories catalog, requesting it by e-mail or by consulting the manufacturer's website.

⚠ CAUTION!

Identification plates are affixed to the rear of the device and on the speakers with the technical data to be communicated to the Authorized Service Center, the Manufacturer or the Dealer in case of request for accessories or spare parts.

⚠ CAUTION!

Replacement diffusers can only be obtained from the selling company. Using a diffuser other than the one indicated can be dangerous.

MAGNETOTHERAPY

Magnetotherapy is a form of physiotherapy that uses electromagnetic energy, stimulating tissue regeneration. Magnetotherapy is part of physical therapy using low frequency and low intensity pulsed magnetic fields; uses the action of a magnetic field induced by the electric current flowing through a coil (solenoid). It is believed that the fundamental principle through which magnetotherapy exerts the therapeutic effects is to restore order in an industry presumably in magnetic disorder. Magnetotherapy also performs an action that promotes tissue repair processes and stimulates the natural organic defenses. Based on the biological effects, the therapeutic action of magnetic fields can be summarized in two main points of attack:

- anti-inflammatory and anti-edema
- stimulating tissue repair processes

Cell membranes are in practice of the "tiny batteries" of which it was possible to measure the voltage delivered, more precisely in healthy nerve cells a potential difference of 90 millivolts is measured between the inner nucleus and the outer membrane (thousandths of a volt), in other cells this voltage is around 70 millivolts. When the tension inside the cells tends to discharge, the body feels the consequences in the form of inflammatory processes, pain in the bones, joints, back, wounds that do not heal. When these cells get sick, due to an infection or trauma or any other cause, they lose their energy reserve, so a cell that, if healthy, should have a voltage of 70 millivolts, only measures 50/55 millivolts; when this voltage falls below 30 millivolts there is necrosis, ie cell death.

The purpose of magnetotherapy is to recharge and regenerate cells lacking vital energy.

The pathology of the musculoskeletal system represents the most specific field of application of magnetotherapy; In particular, the post-traumatic one responds excellently to this type of treatment. All traumatic events, from small speech trauma to severe fracture, can benefit from this therapy with sometimes sensational healing times. The bone pathology treated with magnetotherapy also showed reparative processes and shorter recovery times.

It can therefore be said that magnetotherapy, exploiting high frequencies, can represent a first choice therapeutic method, especially in inflammatory, traumatic and degenerative pathology of the osteo-articular and muscle-tendon systems. Pulsed magnetic fields improve its osteogenesis. They favor the increase of vascular circulation (hypervascularization) and favor the increase in bone strength. It also has beneficial effects on the central - peripheral nervous system and on inflammations, favoring:

- increase of ionic exchanges at the cell membrane level;
- membrane stabilization with hyperpolarization phenomena related to the influence on the sodium / potassium pump and ion exchange;
- normalization of electrical conductivity;
- analgesic effect;
- increase in mental performance and individual performance;
- neuroregulatory action at the level of the diencephalon, the reticular substance, hypothalamus, adrenal gland, liver and spleen;

- anti-edema effect;
- effect on the pump sodium / potassium;
- modification of membrane permeability;
- bacteriostatic activity;
- resolution of muscle spasm;
- analgesic action;
- acceleration of soft tissue healing processes;
- normalization of the potential difference between the injured region of the cell membrane and the normal region (there is a potential imbalance in many pathologies).

CMP-BF (low frequency pulsed magnetic fields)

The use of electromagnetic energy, in the form of low frequency and low intensity pulsed magnetic fields, to stimulate tissue regeneration is now more than consolidated. It is believed that the fundamental principle through which the pulsed magnetic fields BF exert the beneficial effects is that of restoring order in a sector presumably in magnetic disorder. The pulsed magnetic fields BF also carry out an action that favors the reparative processes of the tissues and stimulates the natural organic defenses. Based on the biological effects, the healing action of magnetic fields can be summarized in two main points of attack:

- anti-inflammatory and anti-edema
 - stimulating tissue repair processes
- The purpose of BF pulsed magnetic fields is to recharge and regenerate cells lacking vital energy. The pulsed magnetic fields -BF applied with this apparatus, together with the pulsed light, are particularly effective in the ANTI-AGING TISSUE ACTION:

- action on collagen;
- vasodilation,
- improvement of skin and connective tissue metabolism.

CMP - HF: FIELDS HIGH FREQUENCY PULSED MAGNETICS

The use of electromagnetic energy, in the form of high frequency and very low intensity pulsed magnetic fields, to stimulate tissue regeneration. It is believed that the fundamental principle through which pulsed magnetic fields HF exert beneficial effects is to restore order to a sector presumably in magnetic disorder. The HF pulsed magnetic fields also have an action that favors tissue repair processes and stimulate the natural organic defenses. Based on the biological effects, the healing action of magnetic fields can be summarized in two main points of attack:

- antiflogistics and anti-edema
- stimulating tissue repair processes

The purpose of pulsed HF magnetic fields is to recharge and regenerate cells lacking vital energy. The pulsed HF magnetic fields applied with this apparatus, they are particularly effective in the

ANTIAGING TISSUE ACTION:

- action on collagen;
- vasodilation,
- improvement of skin and connective tissue metabolism.

DIFFERENCE BETWEEN LOW FREQUENCY MAGNET AND HIGH FREQUENCY

Let's spend a few words to better clarify the difference between low frequency (BF) and high frequency (HF) magnetotherapy, since we have been able to see that around there is a lot of confusion in this regard: many believe that the difference between low frequency and high frequency magnetotherapy is due to the fact that the former work with magnetic fields having maximum frequencies between 200 Hz and about 500 Hz while the latter reach higher frequencies, up to 3000 - 5000 Hz and beyond; others consider BF the magneto that work with the frequency of the electrical network (50 Hz) while the others work with magnetic fields at a "higher" frequency, typically from 100 to over 2000 Hz. Let's say immediately that all these devices are to be considered however low frequency, as the distinction between BF systems and HF systems is much deeper and more radical. It is meant by magnetotherapy in Low Frequency any device that generates a variable alternating current having a frequency of variation between a few Hertz and a few thousand Hertz and in which the alternating current thus generated is sent directly to the magnetic diffuser (usually a sort of transformer or ring with many coils of conductor wire) intended to transform the alternating current into an alternating magnetic field having the same frequency as the current that generates it, and to radiate the

magnetic field thus obtained on the parts of the body to be treated. As can be seen, in BF devices the alternating current is sent "directly" to the diffuser which transforms it into a magnetic field. In devices operating at high frequency there is an extra step and the subject changes completely. First of all, there are two frequency generators: one that generates one "Carrier" at frequencies ranging from 5 - 6 Mhz (millions of Hz) up and destined to be sent to the magnetic field diffuser, and another that generates a series of lower frequencies (the same as the BF devices) that they range from a few Hz to several thousand Hz; the latter are used for "Modulate" the high frequencies radiated by the speaker. The final result is a series of pulses very short high frequency (about 100 microseconds in duration) emitted by the generator of HF and which are repeated at the frequency established by the BF generator which in the case of our device can go, depending on the selected therapy, from 1 Hz to 40000 Hz (40 KHz). In essence, the high frequency is used to convey the low frequencies useful in therapy into the tissues. The high frequency has a capacity of penetration into the tissues considerably higher than the low frequency, so it is possible to use very low powers to obtain a remarkable effectiveness. Usually in BF devices there is also the GAUSS control, that is the intensity of the magnetic field that it can even reach a few hundred Gausses. This intensity must be carefully adjusted so as not to exceed the recommended values. In devices in

HF this is no longer necessary, as the magnetic field generated is very weak, even

to a very small fraction of Gauss, given that the radiated powers are negligible, but since, as we said, the high frequencies penetrate more easily

in the tissues, the small power available easily reaches the cells themselves, generating those therapeutic effects that have now been demonstrated for over forty years of studies all over the world.

Furthermore, there is another effect that many ignore

or neglect its importance:

the mixing of the two signals (high frequency carrier of some Mhz and low frequency modulation from a few Hz up to

40 KHz) generates the so-called "Harmonics", that is multiple frequencies of the carrier (double, triple, quadruple, etc.) which can extend up to several hundred Mhz. These harmonics therefore have the ability to trigger Bioresonance phenomena in the tissues, helping to make the cells absorb the energy they need to recharge and thus contributing to the therapeutic action in a way and with such an efficiency that it is difficult for the BFs alone. to do. These harmonics are of very weak intensity, detectable only with very sensitive instruments, but fundamental for our purposes.

As you can well guess, a high frequency magnetotherapy is considerably more complex and expensive to carry out than a low frequency one, so these devices are more difficult to find on the market or even in specialized centers and, usually, much more expensive than those in low frequency.

PHOTOCATALYSIS BLUE AND INFRARED

In recent years, the action of specific lights in skin rejuvenation and tissue biostimulation has been studied extensively and in the treatment of dermal pathologies such as acne, cutaneous precancerous and non-melanoma skin tumors as well as the treatment of painful muscular states, neuralgia, myalgia, atalgia, trophic disorders. The high intensity BLUE light (420 + / - 5nm) causes by means of a photo-oxidation a lowering of the bilirubin in the serum which is decomposed into non-toxic water-soluble products easily eliminated by the kidneys and with consequent photorejuvenation activity; and again it appears to be particularly effective for acne therapy by causing a local photo-exciting reaction with the production of free radicals that destroy the bacteria in the pilosebaceous follicles without damaging the surrounding healthy cells. There are no known side effects. The INFRARED

is a treatment that is performed with an electromagnetic radiation with a wavelength above 780nm, generally using IR frequencies with a wavelength of 780 - 1400nm (IRA) The infrared is transformed into thermal energy on the upper layers of the skin by absorption, only a small part is reflected. Pulsed light is more effective than continuous light as it produces photons in much higher quantities.

Blue light for effective biostimulation on moderate acne, skin aging and to promote healing

Phototherapy uses a selective non-coherent light produced by LEDs. In the most advanced systems it uses

of a plasma light more powerful than the previous ones and with luminous bands capable of interacting with the pathological tissues at several levels.

HOW IT WORKS

Light, in particular intense blue light with infrared, has the ability to selectively destroy tissues, previously treated with a photosensitizing agent (ALA). Blue light together with infrared has proved to be very effective for the treatment of moderate acne, skin aging and for speeding up the reparative processes of post-surgical wounds, peeling, surgical lasers and in the treatment of skin ulcers. It can also be performed without activator (ALA), exploiting the ability of infrared to improve tissue metabolism and blue light to bind the acne bacterium, responsible for inflammatory phases of the disease, reducing it consistently.

INDICATIONS

Acne - Photoaging even advanced with the presence of skin precancerous (actinic keratoses) - Scar outcomes - Ulcers - Skin biostimulation to reduce recovery times after laser treatments, peeling, etc.

SIDE EFFECTS

For phototherapy with the use of a photosensitizer, the cycles are of 2 - 3 sessions at 2 - 4 weeks away for acne, photoaging and cancers other than melanoma. In this case the times of recovery is 2 - 3 days, with redness and sometimes edema of the treated area. Exposure to light is to be avoided for 48 hours. Phototherapy without activator requires cycles of 10 sessions (2 per week) and does not involve any type of post-treatment problem.

Therapeutic Programs

PROGRAMS PATHOLOGY

THERAPEUTIC PROGRAMS

03 PAIN RELIEF
05 ARTHRITIS
06 LUMBOSACRAL ARTHROSIS
08 ASTHENIA
09 MUSCULAR ATROPHY
11 BURSTITIS
12 BRACHIALGIA
16 NECK
18 BRUISES
20 DISTORTIONS
21 FINGERS PAIN
22 WRIST PAIN
23 INTERCOSTAL PAIN
24 INTERCOSTAL PAIN
25 JAW PAIN
27 PAIN OF FOOT UNDERSIDE
28 VERTEBRAL PAIN
29 DORSALGIA
34 EPICONDYLITIS
40 NASAL SEPTUM FRACTURE
41 BONE FRACTURES
46 ACUTE GOUT
51 ISCHIALGIA
58 METATARSALGIA
59 DUPUYTREN'S DISEASE
64 OSTEOPOROSIS
74 RHEUMATISMS
77 LUMBAGO
78 SCIATICA
81 SPONDYLITIS
82 MUSCLE SPRAINS
83 HELL PAIN
84 TENDINITIS
86 STIFF NECK

***i* INFORMATION**


For greater treatment effectiveness, alternate seats with LF diffusers / accessories to those with HF diffusers / accessories.

The duration of all programs is 45 minutes, while only the "universal" program n. 70 does not have a predefined time; the treatment stops when the device is turned off.

PATHOLOGY

PAIN RELIEF


MAGNETIC INTENSITY LF	28 - 32 G
MAGNETIC FREQUENCY LF	74 – 82 Hz
MAGNETIC FREQUENCY HF	500 - 10000 Hz
PROGRAM	03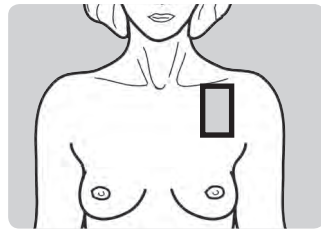

PATHOLOGY

BRACHIALGIA


MAGNETIC INTENSITY LF	35 - 42 G
MAGNETIC FREQUENCY LF	60 – 66 Hz
MAGNETIC FREQUENCY HF	500 - 10000 Hz
PROGRAM	12

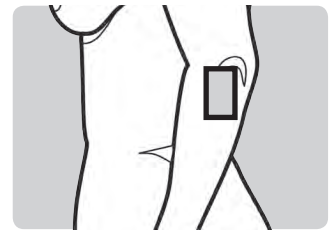PATHOLOGY

ARTHRITIS


MAGNETIC INTENSITY LF	30 - 37 G
MAGNETIC FREQUENCY LF	40 – 52 Hz
MAGNETIC FREQUENCY HF	500 - 5000 Hz
PROGRAM	05

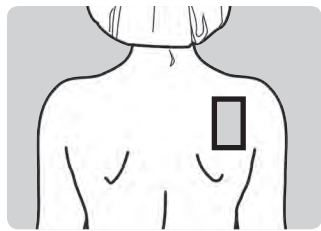PATHOLOGY

NECK


MAGNETIC INTENSITY LF	36 - 43 G
MAGNETIC FREQUENCY LF	42 – 53 Hz
MAGNETIC FREQUENCY HF	500 - 3000 Hz
PROGRAM	16

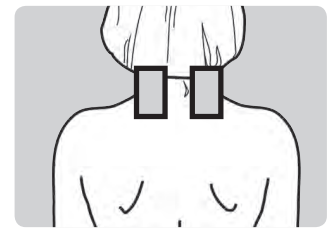PATHOLOGY

LUMBOSACRAL ARTHROSIS


MAGNETIC INTENSITY LF	28 - 34 G
MAGNETIC FREQUENCY LF	20 – 26 Hz
MAGNETIC FREQUENCY HF	1000 - 5000 Hz
PROGRAM	06

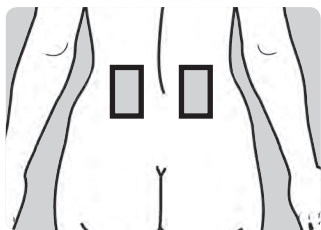PATHOLOGY

BRUISES


MAGNETIC INTENSITY LF	30 - 32 G
MAGNETIC FREQUENCY LF	20 – 26 Hz
MAGNETIC FREQUENCY HF	500 - 10000 Hz
PROGRAM	18

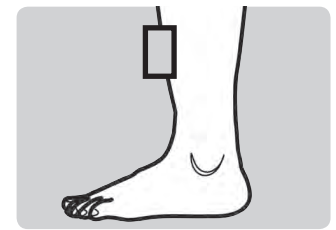PATHOLOGY

ASTHENIA


MAGNETIC INTENSITY LF	30 - 37 G
MAGNETIC FREQUENCY LF	46 – 53 Hz
MAGNETIC FREQUENCY HF	500 - 1000 Hz
PROGRAM	08

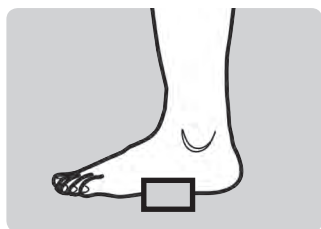PATHOLOGY

DISTORTIONS


MAGNETIC INTENSITY LF	30 - 36 G
MAGNETIC FREQUENCY LF	70 – 76 Hz
MAGNETIC FREQUENCY HF	500 - 10000 Hz
PROGRAM	20

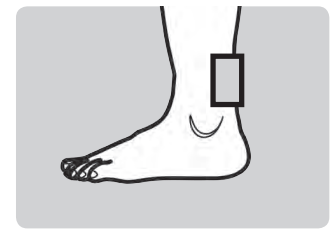PATHOLOGY

MUSCULAR ATROPHY


MAGNETIC INTENSITY LF	30 - 38 G
MAGNETIC FREQUENCY LF	64 – 74 Hz
MAGNETIC FREQUENCY HF	1000 - 5000 Hz
PROGRAM	09

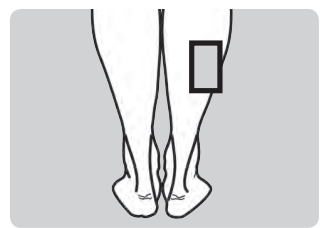PATHOLOGY

FINGERS PAIN


MAGNETIC INTENSITY LF	34 - 42 G
MAGNETIC FREQUENCY LF	15 – 22 Hz
MAGNETIC FREQUENCY HF	1000 - 10000 Hz
PROGRAM	21

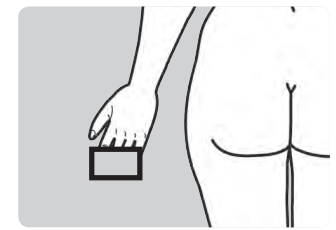PATHOLOGY

BURSITIS


MAGNETIC INTENSITY LF	26 - 32 G
MAGNETIC FREQUENCY LF	88 – 93 Hz
MAGNETIC FREQUENCY HF	1000 - 5000 Hz
PROGRAM	11

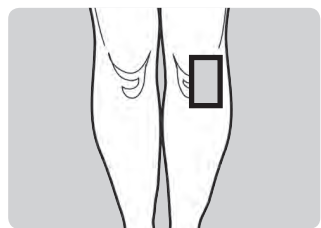PATHOLOGY

WRIST PAIN


MAGNETIC INTENSITY LF	28 - 38 G
MAGNETIC FREQUENCY LF	26 – 32 Hz
MAGNETIC FREQUENCY HF	1000 - 5000 Hz
PROGRAM	22

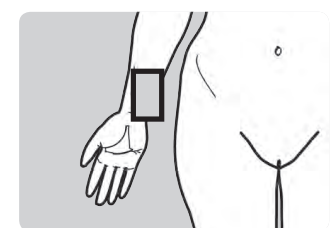PATHOLOGY

INTERCOSTAL PAIN


MAGNETIC INTENSITY LF	36 - 42 G
MAGNETIC FREQUENCY LF	42 - 48 Hz
MAGNETIC FREQUENCY HF	500 - 5000 Hz
PROGRAM	23

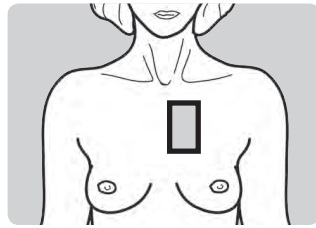PATHOLOGY

EPICONDYLITIS


MAGNETIC INTENSITY LF	20 - 26 G
MAGNETIC FREQUENCY LF	20 - 26 Hz
MAGNETIC FREQUENCY HF	1000 - 10000 Hz
PROGRAM	34

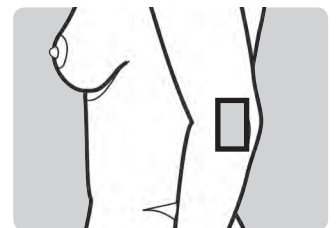PATHOLOGY

INTERCOSTAL PAIN

MAGNETIC INTENSITY LF	27 - 33 G
MAGNETIC FREQUENCY LF	43 - 51 Hz
MAGNETIC FREQUENCY HF	500 - 10000 Hz
PROGRAM	24

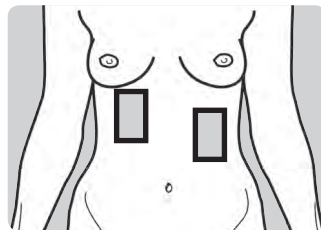PATHOLOGY

NASAL SEPTUM FRACTURE

MAGNETIC INTENSITY LF	36 - 42 G
MAGNETIC FREQUENCY LF	20 - 26 Hz
MAGNETIC FREQUENCY HF	1000 - 5000 Hz
PROGRAM	40

PATOLOGIA

JAW PAIN


MAGNETIC INTENSITY LF	27 - 33 G
MAGNETIC FREQUENCY LF	50 - 57 Hz
MAGNETIC FREQUENCY HF	500 - 10000 Hz
PROGRAM	25

PATOLOGIA

BONE FRACTURES


MAGNETIC INTENSITY LF	25 - 33 G
MAGNETIC FREQUENCY LF	21 - 28 Hz
MAGNETIC FREQUENCY HF	100 - 5000 Hz
PROGRAM	41

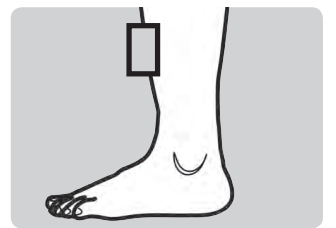PATHOLOGY

PAIN OF FOOT UNDERSIDE


MAGNETIC INTENSITY LF	36 - 42 G
MAGNETIC FREQUENCY LF	40 - 46 Hz
MAGNETIC FREQUENCY HF	500 - 10000 Hz
PROGRAM	27

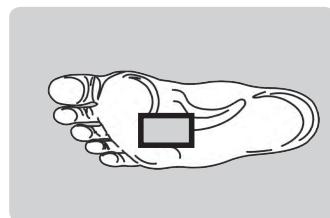PATHOLOGY

ACUTE GOUT


MAGNETIC INTENSITY LF	35 - 42 G
MAGNETIC FREQUENCY LF	30 - 36 Hz
MAGNETIC FREQUENCY HF	1000 - 10000 Hz
PROGRAM	46

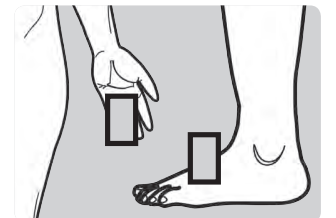PATHOLOGY

VERTEBRAL PAIN


MAGNETIC INTENSITY LF	37 - 42 G
MAGNETIC FREQUENCY LF	78 - 83 Hz
MAGNETIC FREQUENCY HF	1000 - 10000 Hz
PROGRAM	28

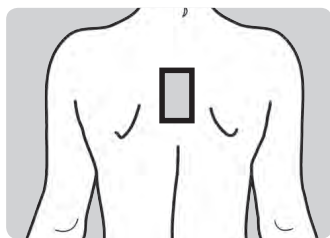PATHOLOGY

ISCHIALGIA


MAGNETIC INTENSITY LF	32 - 38 G
MAGNETIC FREQUENCY LF	38 - 40 Hz
MAGNETIC FREQUENCY HF	1000 - 5000 Hz
PROGRAM	51

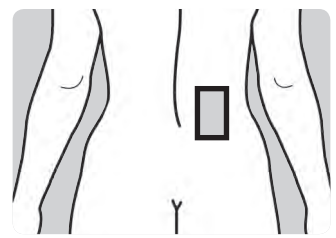PATHOLOGY

DORSALGIA


MAGNETIC INTENSITY LF	38 - 42 G
MAGNETIC FREQUENCY LF	20 - 25 Hz
MAGNETIC FREQUENCY HF	1000 - 5000 Hz
PROGRAM	29

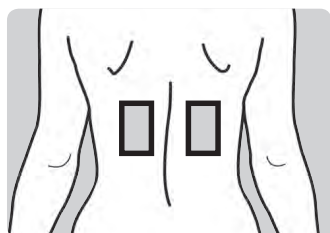PATHOLOGY

METATARSALGIA


MAGNETIC INTENSITY LF	31 - 37 G
MAGNETIC FREQUENCY LF	20 - 27 Hz
MAGNETIC FREQUENCY HF	500 - 10000 Hz
PROGRAM	58

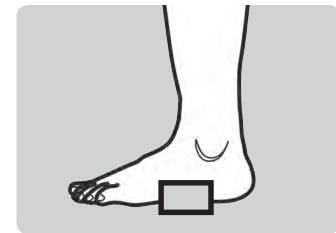PATHOLOGY

DUPUYTREN'S DISEASE


MAGNETIC INTENSITY LF	38 - 44 G
MAGNETIC FREQUENCY LF	20 - 26 Hz
MAGNETIC FREQUENCY HF	1000 - 10000 Hz
PROGRAM	59

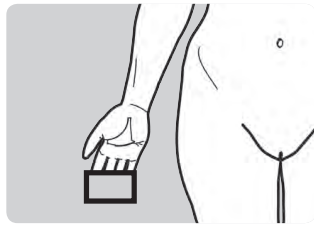PATHOLOGY

SPONDYLITIS


MAGNETIC INTENSITY LF	36 - 40 G
MAGNETIC FREQUENCY LF	40 - 46 Hz
MAGNETIC FREQUENCY HF	1000 - 10000 Hz
PROGRAM	81

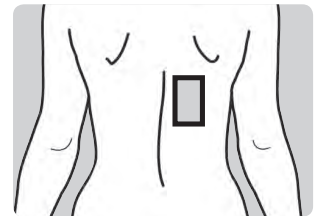PATHOLOGY

OSTEOPOROSIS


MAGNETIC INTENSITY LF	30 - 36 G
MAGNETIC FREQUENCY LF	16 - 22 Hz
MAGNETIC FREQUENCY HF	500 - 5000 Hz
PROGRAM	64

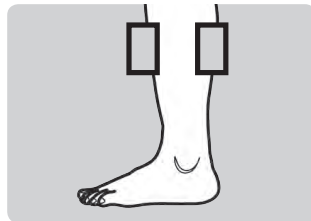PATHOLOGY

MUSCLE SPRAINS


MAGNETIC INTENSITY LF	40 - 42 G
MAGNETIC FREQUENCY LF	20 - 26 Hz
MAGNETIC FREQUENCY HF	1000 - 5000 Hz
PROGRAM	82

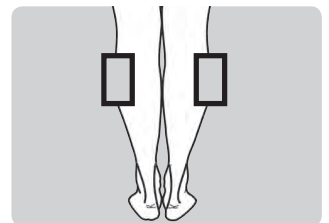PATHOLOGY

RHEUMATISMS


MAGNETIC INTENSITY LF	37 - 41 G
MAGNETIC FREQUENCY LF	88 - 94 Hz
MAGNETIC FREQUENCY HF	1000 - 10000 Hz
PROGRAM	74

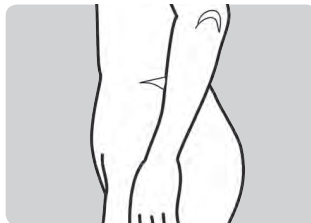PATHOLOGY

HELL PAIN


MAGNETIC INTENSITY LF	28 - 32 G
MAGNETIC FREQUENCY LF	44 - 53 Hz
MAGNETIC FREQUENCY HF	500 - 5000 Hz
PROGRAM	83

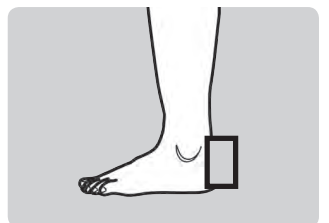PATHOLOGY

LUMBAGO


MAGNETIC INTENSITY LF	35 - 44 G
MAGNETIC FREQUENCY LF	40 - 46 Hz
MAGNETIC FREQUENCY HF	1000 - 5000 Hz
PROGRAM	77

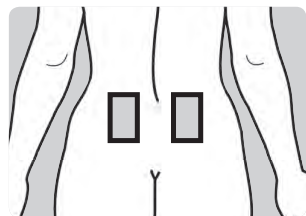PATHOLOGY

TENDINITIS


MAGNETIC INTENSITY LF	36 - 42 G
MAGNETIC FREQUENCY LF	70 - 76 Hz
MAGNETIC FREQUENCY HF	1000 - 10000 Hz
PROGRAM	84

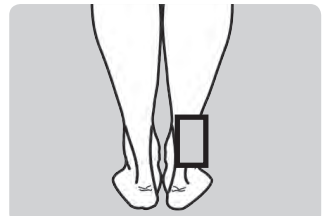PATHOLOGY

SCIATICA


MAGNETIC INTENSITY LF	19 - 27 G
MAGNETIC FREQUENCY LF	88 - 93 Hz
MAGNETIC FREQUENCY HF	1000 - 10000 Hz
PROGRAM	78

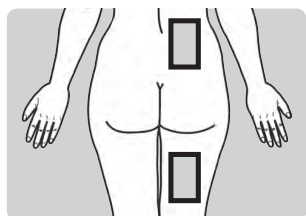PATHOLOGY

STIFF NECK

MAGNETIC INTENSITY LF	28 - 32 G
MAGNETIC FREQUENCY LF	50 - 56 Hz
MAGNETIC FREQUENCY HF	500 - 3000 Hz
PROGRAM	86

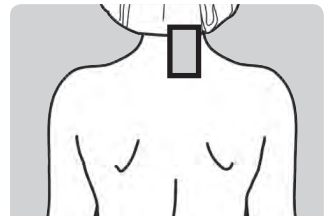QUESTIONS AND ANSWERS**❗ INFORMATION****When can the appliance be used?**

The device can be used at any time of the day. We recommend that you take a moment to devote yourself exclusively to the treatment, without having to worry about anything else.

Can Magneto-light treatments be harmful if used often?**⚠ CAUTION!**

Magneto-light applications do not create any harm except in contraindicated cases (see the "Contraindications" chapter). However, it is advisable to follow the instructions, avoiding excessive actions or too frequent applications.

If you increase the potency, do you get more results?**⚠ CAUTION!**

NO. The APPLIANCE is equipped with specific programs which already foresee the optimal level of power to be used.

How long does it take to see the first results?

The effects are sometimes immediate. Much depends on the pathology and sensitivity of the treated areas.

Can multiple treatments be done on the same area several times a day?**What is the maximum recommended daily usage time?**

One of the advantages of Magnetotherapy is that it has no particular contraindications or limits in use. We suggest,

however, a daily use on each body area.

How can I clean the treatment diffuser after each application?

Cleaning the diffuser is important, proceed as indicated in the "Cleaning" chapter.

If the treatment diffuser has cracks or the cable appears damaged, can I still use them?**⚠ CAUTION!**

Absolutely no! After each treatment it is necessary to check the treatment diffuser and the cable as indicated in the "Maintenance" chapter.

Can I have applications on the lower abdomen?**⚠ CAUTION!**

These applications are absolutely contraindicated and not recommended in case of PREGNANCY, (see the "Contraindications" chapter).

I have a pacemaker, can I use THE DEVICE?**⚠ CAUTION!**

NO, use is contraindicated of electronic devices in general to people with cardiac pacemakers and also in the presence of implanted electrodes. A doctor should be consulted (see chapter "Against indications").

TROUBLESHOOTING**⚠ CAUTION!**

No intervention on the equipment is permitted, as in addition to generating possible dangerous situations, it will invalidate the validity of the Warranty.

What to do if the appliance does not show any sign of operation when the power switch is turned on?

- check that the power cable is properly inserted in its housing and in the socket.
- check that the power cable does not show signs such as:
 - breaks or interruptions;
 - deterioration.
- check the integrity of the fuse inside the fuse holder.

⚠ CAUTION!

In case of breakage or deterioration of the power cable, immediately disconnect it from the power socket and replace it immediately.

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© 2019 HSD S.r.l. RSM - Edizione Febbraio 2019 - Rel. 0.0